

THIS RESIDENT HAS A BED ALARM

Setting up the Sensatec Fall Alert System

- 1) The pad has a limited life, so please record the in-service date in the area provided on the label. Please ensure pads are stored in a flat position prior to use.
- 2) Power the control unit by connecting the plug-in power supply and/or 9v battery.
- 3) Place the bed pad over the width of the bed. Position the pad under the buttocks and lower back of the resident. The lower edge should be at the top of the resident's thigh.
Or, position the Sensor Pad so that it will rest under the shoulders and back of the resident. This placement is recommended when early detection is required.
- 4) Insert the Sensor Pad plug into the sensor jack on the Control Unit.

When weight is applied to the Sensor Pad, the Control Unit beeps once to indicate the Control Unit is monitoring and flashes the Green Monitor LED periodically.

- 5) Set the delay time (seconds of removed weight before alarm condition occurs) by pressing the delay button repeatedly to the desired setting.
- 6) Set the localized volume level to the desired setting by pressing the volume button repeatedly until the desired level found.

To respond to an alarm

- 1) Assist the resident into the bed or chair.
- 2) The Fall Alert System will automatically reset and begin monitoring again (you should get a RESET ALARM page to confirm this). *Or*: You'll see the Green Monitor LED flashing to indicate the Control Unit is monitoring.
- 3) If the resident is not available to put back on the pad, then press the SILENCE/HOLD button ONCE to suspend the system until the resident is able to return. This will cancel the alarm condition. The system will automatically start monitoring when weight is re-applied.

To PAUSE / HOLD the System (to move a resident from the bed without alarming)

- 1) Press the Silence/Hold button ONCE on the Control Unit. (The yellow LED light will flash).
- 2) Assist the resident out of the bed or chair.
- 3) When you put the resident back on the pad, the system will automatically restart monitoring. **OR**; if the yellow LED is flashing, press the Silence/Hold button once to restart the monitoring.